

## HOW TO ASSEMBLE YOUR



### TRANSFER BENCH


Item # 9071

Capacity not to exceed 300 lb



#### WHAT'S INCLUDED

- a. (1) Bench
- b. (1) Backrest
- c. (1) Handle
- d. (2) Adjustable legs
- e. (2) Adjustable suction legs

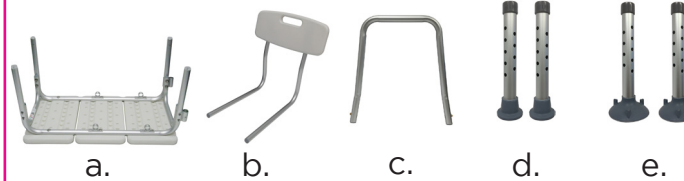


**NO tools required**  
**Quick assembly**

**Follow these**  
**SIMPLE STEPS**

- 1 Place transfer bench upside-down on a flat surface. Depress push buttons on each adjustable suction leg and slide onto the bench frame on the side where the backrest slots are located (see **Figure 1**).

**NOTE:** Suction leg side of bench goes into bathtub. Leg heights may vary depending on the difference in height of bathtub and the floor.

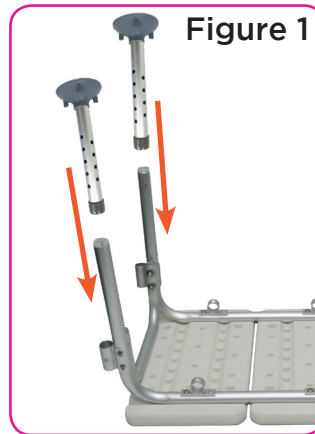


**Figure 1**

- 3 Set transfer bench upright on its legs. Depress push buttons on handle, and insert into the slots on the side of the bench. The handle should angle away from the bench. (see **Figure 3**).



**Figure 3**


- 2 Depress push buttons on each adjustable leg and slide onto the bench frame on the opposite side (see **Figure 2**). Adjust all legs so the bench is parallel to the floor, and ensure all push buttons are engaged before using.

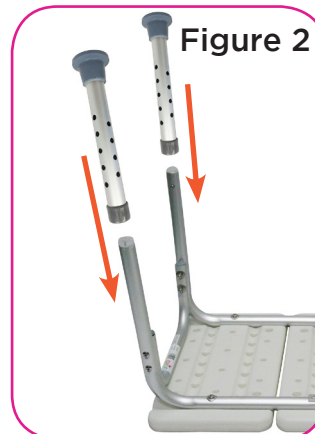

**Figure 2**

- 4 Turn transfer bench onto its side. Slide the backrest into the backrest slots and secure by engaging the push buttons (see **Figure 4**).

The backrest can be attached to either side of bench, depending of which way user would like to face.

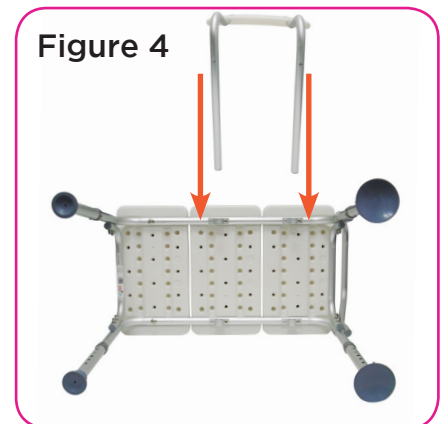

**Figure 4**

