

Shower Stool

SST-S
(DC108 - RevA)

healthcraftproducts.com

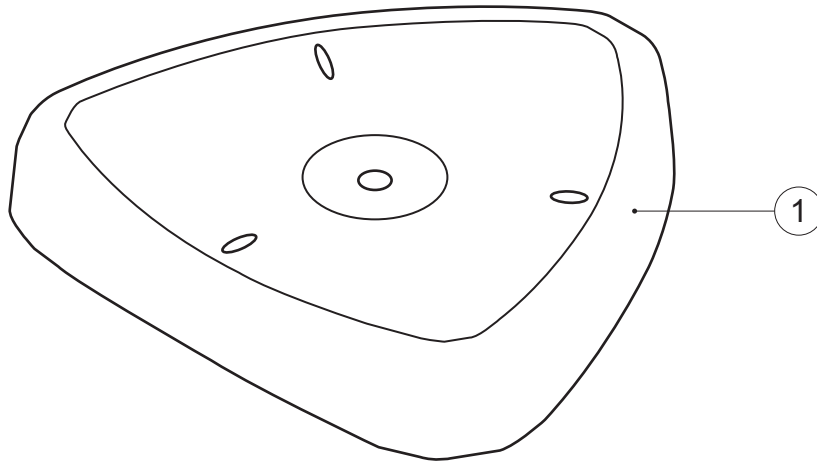
2.00lbs / 0.91kgs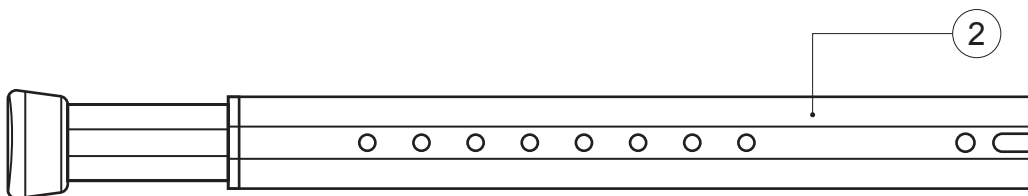


0.55lbs / 0.25kgs


1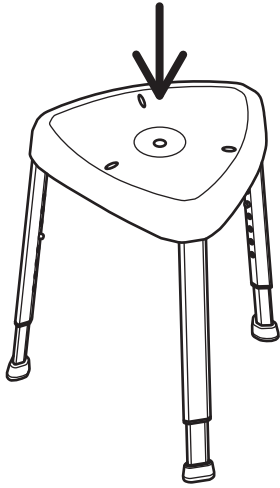


2

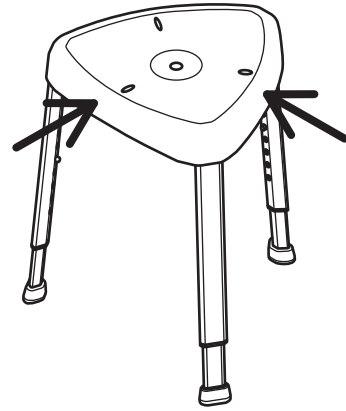
3

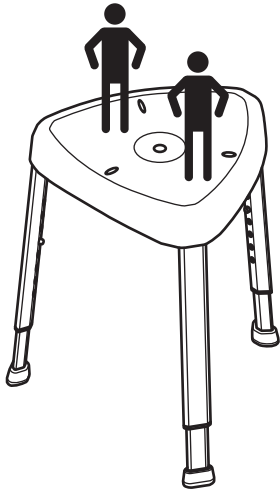
4

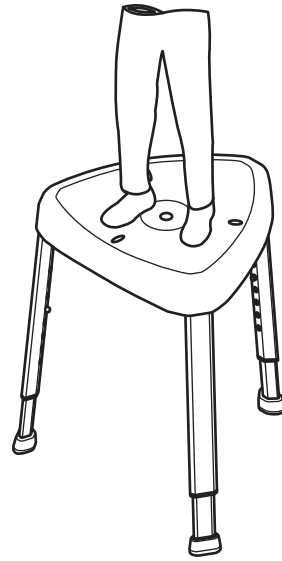5

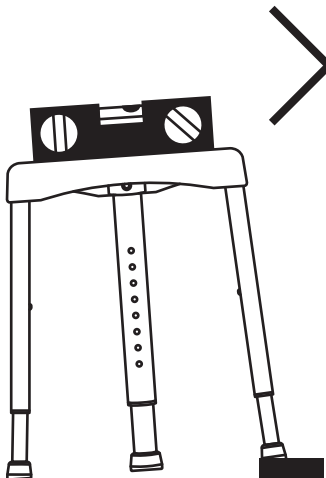
6

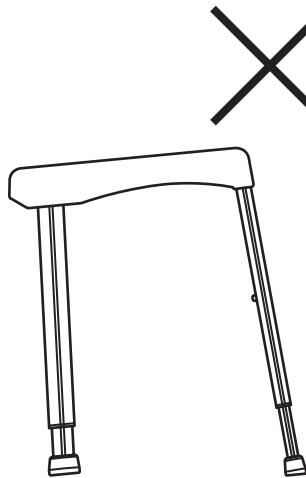
7


1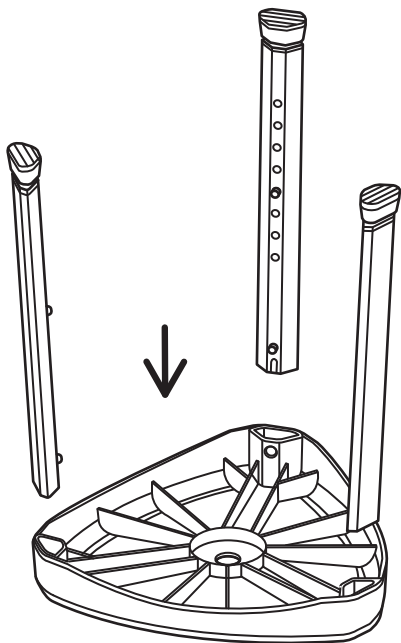


2

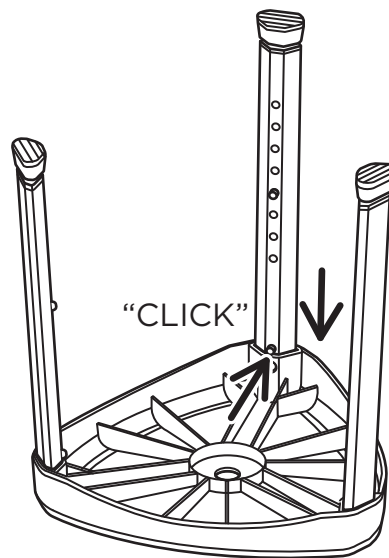
3

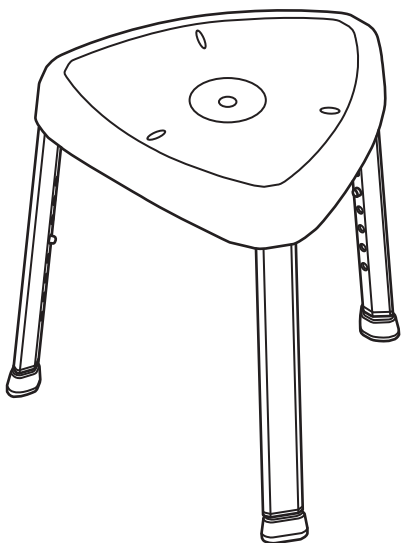4


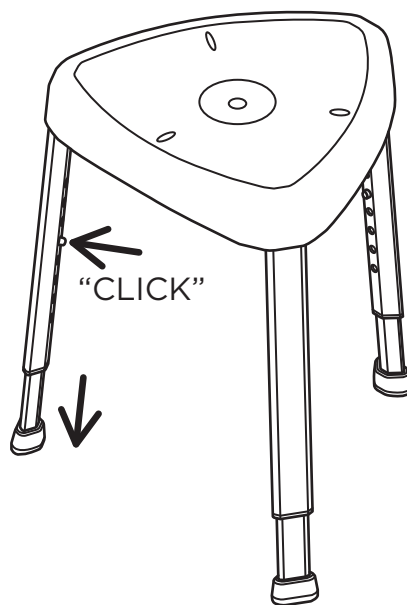
D


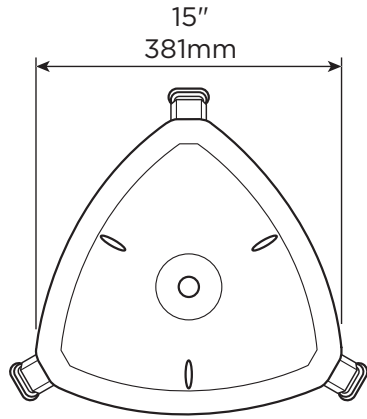
MAX
300lbs
136kg

16.625-23.625"
422-600mm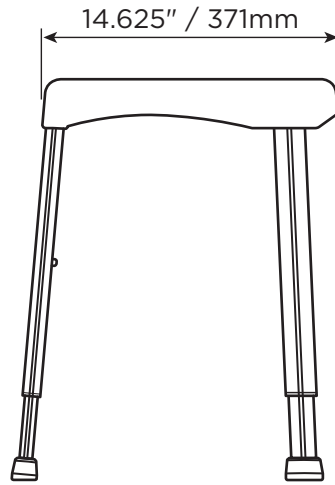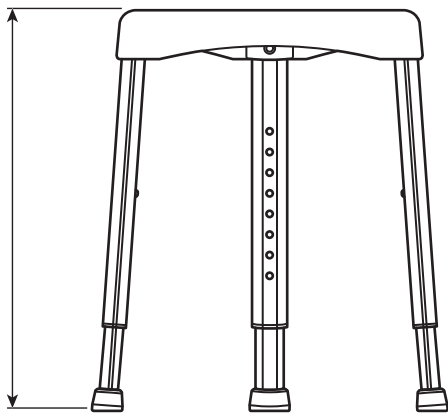

E

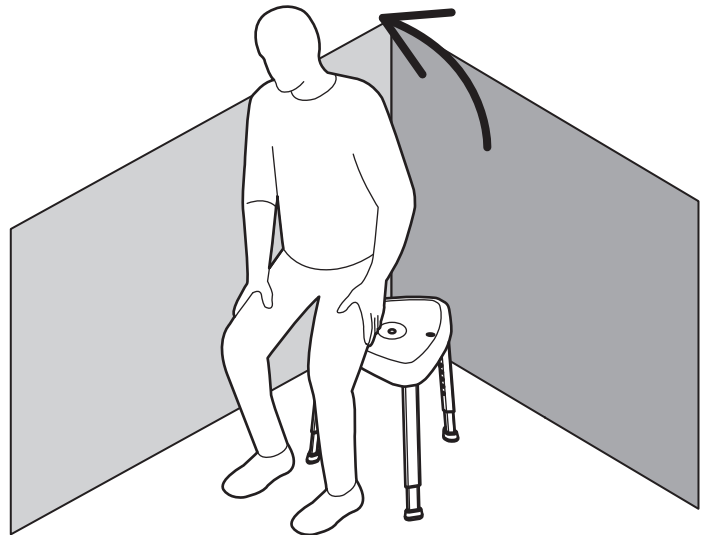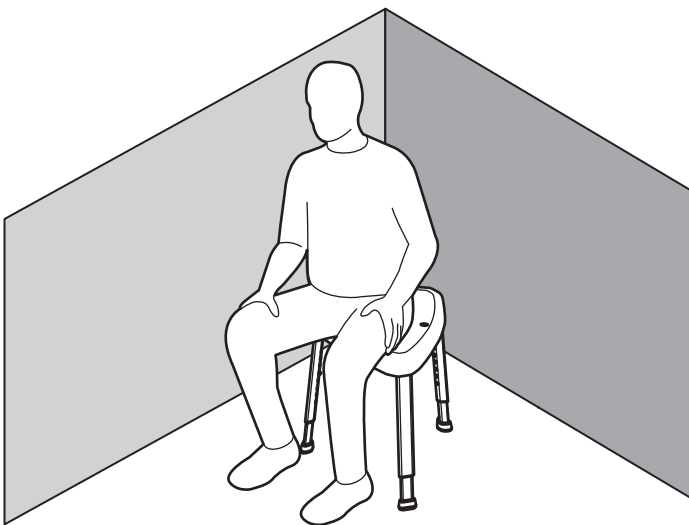English

OVERVIEW

Thank you for choosing the Shower Stool from HealthCraft. Please read and understand the instructions in this manual; keep manual for future reference. It is your responsibility to see that your Shower Stool is properly assembled, installed, and cared for. Failure to follow instructions in this manual could result in serious injury or death. If you are not equipped to undertake the outlined work, we would recommend that you have your Shower Stool installed by a qualified contractor.

The Shower Stool is a high quality, commercial grade shower seat. The Shower Stool is intended to provide seating support for people with reduced mobility. The maximum user weight for this product is 300lbs / 136kgs. Note: Product is not intended to support full body weight (i.e. do not stand on the bench). The Shower Stool is not to be used in any other way than described above.

ONE YEAR WARRANTY

Products are covered by a one year warranty against defects in materials and workmanship for the original purchaser. Warranty excludes products that have been damaged through misuse, accidental damage, alteration, normal wear and tear, wood material and stain, or the use of corrosive or abrasive cleaning products.

Buyer hereby indemnifies, agrees to hold harmless and defend HealthCraft Products Inc. from and against any and all liabilities, claims, (founded and unfounded), losses, damages, costs and expenses (including without limitation consequential damages and reasonable professional fees) resulting from buyers specification, application, or improper use of goods described herein; buyers omission or neglect. HealthCraft Products Inc. does not assume any liability for damage resulting from services performed by others or faulty installation, misuse or misapplication of goods sold by HealthCraft Products Inc. HealthCraft Products Inc. shall not be liable for prospective profits or special, indirect, or consequential damages, or for the cost of any corrective work done without HealthCraft Products Inc. prior approval. HealthCraft Products Inc. total liability hereunder shall in no event exceed the purchase price of the goods specified hereon.

Specifications subject to change without notice.

HealthCraft Products Inc.
2790 Fenton Road
Ottawa, Canada
K1T 3T7

Gate 88
Kanalgatan 45B
Skellefteå, Sweden
93123

DELIVERY CONTENTS / PARTS

FIGURE A.

1. Plastic Seat
2. Leg (x3)

INSTALLATION WARNINGS

FIGURE B.

1. Stool is intended to be used as a sitting surface only (i.e. downward vertical force only).
2. CAUTION - Do not exert sideward forces on the stool.
3. CAUTION - Do not use stool for more than 1 person at a time.
4. CAUTION - Do not use stool for purposes other than sitting (i.e. do not stand on the bench).
5. CAUTION - Do not use stool on uneven surfaces that may cause stool to be unstable.
6. CAUTION - Ensure all legs are fully engaged and set to the same length.
7. CAUTION - Stool and bathtub are slippery when wet.

ASSEMBLY

FIGURE C.

1. Slide leg into receptacle on the bottom of plastic seat.
2. Push locking pin and slide leg down until fully engaged.
3. Flip seat upright.
4. Push locking pin and set leg height. Ensure all legs are set to the same length.

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