

Welcome to the world of massage and heat!

## OWNER'S MANUAL and OPERATING INSTRUCTIONS

### Massage and Heat System



## IMPORTANT SAFETY CONSIDERATIONS

Please read and follow all safety & warning instructions provided below and on the system before using & connecting to the power source. Failure to comply with safety instructions may result in fire, electrical shock, or personal injury and may damage or impair protection provided by the equipment. Please save all safety instructions.

### NOTICE!

#### To reduce the risk of potential product damage:

- Follow these instructions to help ensure product quality over the life of the system. Failure to follow these instructions may invalidate the warranty.
- Do not attempt to disconnect the power cord by pulling the power cord. Always hold the power plug and pull it out.
- Only use replacement parts specified by your dealer. Unauthorized substitutions may result in fire, electrical shock or injury and will void the warranty. Only use the power supply that accompanied this system.
- If there is a malfunction of any kind, please contact your dealer for further instructions, and for returning it for examination, repair or replacement.

### WARNING!

#### To reduce the risk of potential serious injury to user:

- To reduce the risk of fire or electric shock, do not expose this product or any component of this product to rain or moisture.
- Do not spill liquid on the system. Spilled liquid may damage the system.
- Do not reach for any portion of the system that has fallen into water or any other liquid. Unplug immediately. Contact dealer for service.
- Use the power cord provided only. Do not overload wall outlets. The power outlet used to power this product should be readily accessible for fast disconnection in case of emergency.
- Unplug the power supply should you find it necessary to clean and or move the product. Strongly recommend using only a clean dry cloth.
- Recommend unplugging the system when it will not be used for long periods of time.
- Keep all cords away from heated surfaces and pinch points.
- Any changes or modifications to your system, or use of unauthorized accessories to your system not expressly approved by the manufacturer may be dangerous and void any applicable warranties.
- Close supervision is recommended when this product is used by, on or near children, invalids, or disabled persons. Always check the skin in contact with the heated area of the product (if heat function included).
- Do not use outdoors. FOR INDOOR USE ONLY.

**OPERATE  
YOUR  
SYSTEM  
ACCORDING  
TO THIS  
MANUAL**

**USE ONLY  
THE POWER  
DEVICE  
SUPPLIED  
WITH THIS  
PRODUCT**

### MAINTENANCE INSTRUCTIONS:

For additional assistance please refer to the contact information in your product's owners manual.

# OPERATING INSTRUCTIONS

## **POWER**

Activates Massage System

Press PWR button one time :

\* Lumbar massage motors start working on SELECT mode in medium intensity and medium speed, PWR, 1, SELECT buttons' backlight turn to green, indicator bars for INTENSITY and SPEED will be lighted.

Press PWR again :

\* Buttons' backlight in ZONES area turn to blue, and will turn to low light level in 10 seconds

## **ZONES\***

1 - Lumbar

2 - Seat

Press 1 or 2 to turn on massage motors, backlight turning to green, press again to stop massage motors.

massage motors will automatically turn off after 15 mins working

## **INTENSITY**

Controls INTENSITY of the massage pulse transducers while in any function. Momentary presses will generate a slight change. Holding down either the + (UP) or - (DOWN) button will achieve a greater increase or decrease.

## **SPEED**

Controls the rate at which the massage action travels. The Speed feature operates in the WAVE and PULSE modes. Momentary presses will generate a slight change. Holding down either + (FAST) or - (SLOW) button will achieve a greater increase or decrease.

## **HEAT**

Activates Heat option.

\* Press LO: Low heat. (Amber Light)

\* Press HI: High heat. (Red Light)

\* Press either again to turn off.

Works with massage or independently.

30 minute Timer

To continue after Timer shuts off, press either LO or HI button again.

## **MODES**

### **SELECT -**

Lets you choose any or all zones for massage.

\* Press SELECT.

\* Press any or all Zones (1 or 2) to activate.

\* Press any or all Zones (1 or 2) to cancel.

### **WAVE -**

Massage motors move up and down in a wave-like action.

\* Press WAVE.

\* Press any or all Zones (1 or 2) to activate.

\* Press any or all Zones (1 or 2) to cancel.


### **PULSE -**

Massage motors turn On and Off to simulate a pulsing action.

\* Press PULSE.

\* Press any or all Zones (1 or 2) to activate.

\* Press any or all Zones (1 or 2) to cancel.

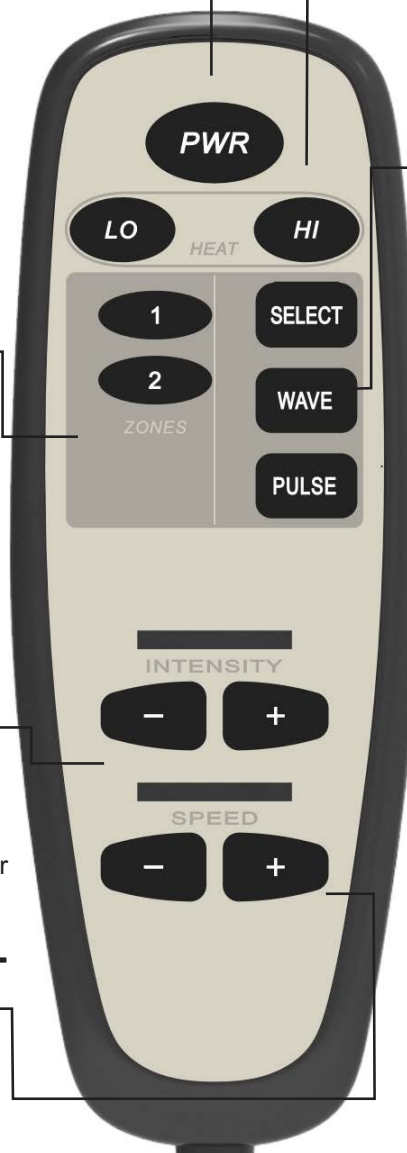

**SAVE THESE INSTRUCTIONS!**